

VICTORIS
HEALTH
2025



Product Catalogue

ENZYSCRUB PRE-SOAKED SPONGE



EnzyScrub Pre-Soaked Sponge

Efficient Pre-Cleaning for Endoscopes and Surgical Instruments

Pre-Saturated with 30 mL ANIOS ANIOSYME 5 SYNERGY Multi-Enzymatic Detergent

EnzyScrub Pre-Soaked Sponge is designed for the immediate pre-cleaning of endoscopes and surgical instruments, ensuring optimal removal of organic residues before thorough reprocessing. Pre-saturated with 30 mL ANIOS ANIOSYME 5 SYNERGY, this multi-enzymatic sponge enhances the breakdown of proteins, lipids, and carbohydrates for superior cleaning efficiency.

Key Features:

- ✓ Ready-to-Use Convenience – Pre-saturated with a powerful multi enzymatic detergent.
- ✓ Enhanced Cleaning Power – Effectively removes organic contaminants to support optimal instrument reprocessing.
- ✓ Single-Use for Infection Control – Reduces the risk of cross-contamination by ensuring a fresh sponge for each device.
- ✓ Time-Saving & Efficient – Quick activation with water optimizes workflow in busy medical settings.
- ✓ Lint Free – Latex Free - Dust Free

How to Use:

1. Open the package and remove the pre-saturated sponge.
2. Immerse in water and squeeze 3-5 times to activate the multi-enzymatic detergent.
3. Gently squeeze out excess liquid.
4. Wipe the exterior surfaces of the endoscope or surgical instrument until clean.
5. Discard the sponge after use to prevent cross-contamination.

Safety Precautions:

- ⚠ Protective Measures – Wear appropriate protective eyewear or face shields.
- ⚠ Eye Contact – If contact occurs, rinse thoroughly with water for several minutes. Remove contact lenses if possible and seek medical assistance immediately.
- ⚠ Infection Control – Use one sponge per device. Dispose of after cleaning to prevent cross-contamination.



Product Code: ES-10-1
QTY: 1 piece